


Read this package insert completely before use and comply with the instructions for use.

# HealiAid<sup>®</sup> Collagen Wound Dressing

## GRAPHICAL SYMBOLS

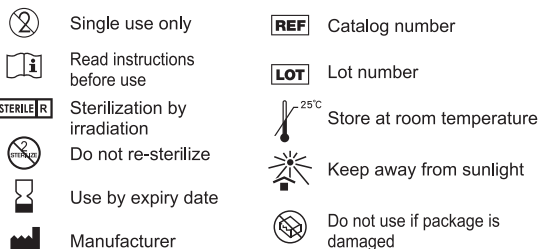

**R**Only Caution:Federal law(USA) restricts this device to sale by or on the order of a licensed healthcare practitioner

## Description

HealiAid<sup>®</sup> Collagen Wound Dressing is a white, porous, pliable and absorbable collagen wound dressing. It is fabricated by fibrous collagen matrix, in which the collagen is purified from bovine Achilles tendons. HealiAid<sup>®</sup> Collagen Wound Dressing is pliable and can be applied easily to clean wounds. HealiAid<sup>®</sup> Collagen Wound Dressing is supplied sterile, non-pyrogenic, and individually packaged. HealiAid<sup>®</sup> Collagen Wound Dressing is for single use only.

## Indications

HealiAid<sup>®</sup> Collagen Wound Dressing is indicated for use in patients who have surgical wounds, donor site/grfts, podiatric wounds, wound dehiscence, traumatic wounds, abrasions, lacerations, partial thickness burns or skin tears. HealiAid<sup>®</sup> Collagen Wound Dressing can be applied to wounds with depth less than 0.3 cm.

## Instructions for use

- Apply HealiAid<sup>®</sup> Collagen Wound Dressing to cover wound site:
  - Clean wounds with appropriate wound cleanser.
  - HealiAid<sup>®</sup> Collagen Wound Dressing should be applied directly on the surface of the wound site. If the wound is dry (with drainage), HealiAid<sup>®</sup> Collagen Wound Dressing can be moistened before applying to the wound.
  - Cover the wound with appropriate cover dressing.
  - Use HealiAid<sup>®</sup> Collagen Wound Dressing products until the wound is fully healed.
- Removal of HealiAid<sup>®</sup> Collagen Wound Dressing:
  - Remove the cover dressing and any remaining HealiAid<sup>®</sup> Collagen Wound Dressing carefully. In the case when HealiAid<sup>®</sup> Collagen Wound Dressing adheres to the wound site, rinse the wound site with normal saline before peeling off. Remove HealiAid<sup>®</sup> from the wound site carefully.
  - The wound site should be cleaned with normal saline before replacing HealiAid<sup>®</sup> Collagen Wound Dressing. Dry the wound site before applying a new HealiAid<sup>®</sup>.
  - Repeat replacing new HealiAid<sup>®</sup> Collagen Wound Dressing products to the wound dressing when needed.

## Precautions

- HealiAid<sup>®</sup> Collagen Wound Dressing should not be used on infected wounds.
- Clean the wound site carefully before applying HealiAid<sup>®</sup> Collagen Wound Dressing.
- HealiAid<sup>®</sup> Collagen Wound Dressing is not intended to use in patients with abnormal coagulation.
- The safety of using HealiAid<sup>®</sup> Collagen Wound Dressing on pregnant women is not clinically evaluated. Surgeon should evaluate the application of HealiAid<sup>®</sup> Collagen Wound Dressing on pregnant women.
- Patients with severe allergy history or hypersensitive to bovine collagen are not recommended to use HealiAid<sup>®</sup>.
- HealiAid<sup>®</sup> Collagen Wound Dressing is contraindicated to patients undergoing desensitization injections to meat products, as these injections may contain bovine collagen.
- Check the expiration date of individual HealiAid<sup>®</sup> before use. Do not use any expired products.
- Do not use HealiAid<sup>®</sup> Collagen Wound Dressing when packaging is damaged, as the sterility of the product cannot be assured.
- HealiAid<sup>®</sup> Collagen Wound Dressing is restricted to single use only. Do not re-sterilized or re-used HealiAid<sup>®</sup> Collagen Wound Dressing.

## Specifications

Product type	Specification
<b>HealiAid<sup>®</sup>-Sheet</b>	
SA152005	15 mm x 20 mm x 0.5 mm
SA203005	20 mm x 30 mm x 0.5 mm
SA253005	25 mm x 30 mm x 0.5 mm
SA304005	30 mm x 40 mm x 0.5 mm
SA101005	100 mm x 100 mm x 0.5 mm
SA204030	20 mm x 40 mm x 3.0 mm
SA303030	30 mm x 30 mm x 3.0 mm
SA405030	40 mm x 50 mm x 3.0 mm
SA505030	50 mm x 50 mm x 3.0 mm
SA100100	100 mm x 100 mm x 3.0 mm
<b>HealiAid<sup>®</sup>-Particle</b>	
SA-G01	1 cc
SA-G02	2 cc
SA-G05	5 cc
SA-G10	10 cc
SA-G20	20 cc

## Storage

HealiAid<sup>®</sup> Collagen Wound Dressing should be stored below 25°C (77°F) in a dry place. Do not freeze or expose HealiAid<sup>®</sup> to extreme conditions.

## Warnings

HealiAid<sup>®</sup> Collagen Wound Dressing cannot be autoclaved because the collagen fibrous structure as well as the packaging will be damaged. HealiAid<sup>®</sup> Collagen Wound Dressing should not be re-sterilized. Residual portions of HealiAid<sup>®</sup> Collagen Wound Dressing should be discarded after use.

HealiAid<sup>®</sup> Collagen Wound Dressing contains bovine collagen and must not use in patients with a history of allergies to any bovine collagen products to prevent hypersensitivity. Hypersensitivity reactions reported when using bovine collagen products including erythema, swelling, induration, and/or urticaria.

**MBi** MAXIGEN BIOTECH INC.

MAXIGEN BIOTECH INC.  
No.88, Keji 1st Rd., Guishan District, Taoyuan City 33383, Taiwan  
TEL : +886-3-3287-222 FAX : +886-3-3287-333

E-MAIL : service@mbi-bio.com

21H2B0022